

PliaFX[®] Prime

Moldable Demineralized Fibers



Optimized Handling. Uncompromised Performance.


What is PliaFX Prime?

PliaFX Prime is 100% bone fibers, demineralized to encourage bone formation and healing. The fibers interlock, allowing the graft to become moldable upon rehydration without the use of a carrier.¹



What is the PliaFX Prime advantage?


100% Bone grows more bone than DBMs containing a carrier, as demonstrated in literature.^{2,3} First-generation DBM putties contain a carrier, such as glycerol, solely to improve handling characteristics of the graft. The proportion of bone content in first-generation DBM putties can be as low as 17% by weight.⁴ LifeNet Health's moldable fiber technology eliminates the need for a carrier, providing 100% bone.

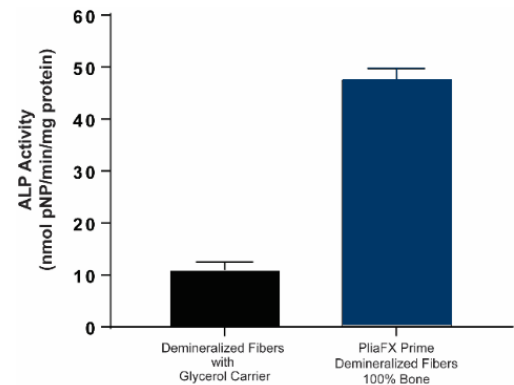

Alkaline phosphatase (ALP) is a marker of the early stages of new bone formation

What makes PliaFX Prime versatile?

PliaFX Prime is primarily used as a standalone graft and can be used in combination with other biomaterials such as autograft, allograft, and/or fluid of surgeon's choice. The precision-machined fibers are designed to interlock with these biomaterials to improve their handling characteristics.⁵

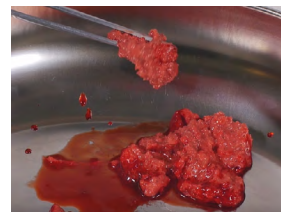

Simulated autograft



Simulated autograft mixed with PliaFX Prime



PliaFX® Prime

Ambient Storage*

Order Code	Volume	Shelf Life
BL-1800-00	0.5 cc	4 years
BL-1800-01	1.0 cc	4 years
BL-1800-02	2.5 cc	5 years
BL-1800-05	5.0 cc	5 years
BL-1800-10	10.0 cc	5 years

*While ambient room temperature has not been defined by regulatory bodies, LifeNet Health would recommend storage at 2°C to 37°C with excursions of less than 24 hours up to 40°C. If an excursion outside this range occurs, please contact LifeNet Health.

How is PliaFX Prime ordered?

All PliaFX Prime orders are placed with LifeNet Health directly by phone: 1-888-847-7831 or email: orders@lifenethealth.org. Business hours are Monday through Friday, from 7am to 8pm EST.

How is PliaFX Prime shipped?

PliaFX Prime is shipped for second day delivery (by 10:30am), free of charge. Next day delivery is also available for a fee.

References

1. Data on file LifeNet Health ES-17-090 Rehydration and Irrigation Characterization for Moldable Demineralized Fibers
2. Data on file LifeNet Health TR-0446 Carrier-Free DBM Fibers (PliaFX Prime) Demonstrate Enhanced Osteoinductive Potential In Vitro Compared to a Similar Glycerol Carrier-Containing Counterpart
3. Boyan BD, Ranly DM, McMillan J, et al. Osteoinductive Ability of Human Allograft Formulations. J Periodontol. September 2006
4. Kay JF, Vaughan LM. Proportional osteoinduction of demineralized bone matrix graft materials. February 2004: AW-0204.1
5. Data on file LifeNet Health ES-16-085 Initial Feasibility for Characterization of Next Generation Fiber Putty

LifeNetHealth.org

LifeNet Health, the LifeNet Health logo, and PliaFX are registered trademarks of LifeNet Health. The third party trademarks used herein are the trademarks of their respective owners.

©2020 LifeNet Health, Virginia Beach, VA. All rights reserved.
68-40-365.01

©DePuy Synthes 2020. All rights reserved.
143484-200615 DSLS
www.jnjmedicaldevices.com

Bench/laboratory testing may not be indicative of clinical outcomes.

Please refer to the instructions for use for a complete list of indications, contraindications, warnings and precautions.

